SaddleLoop®
Retract-O-Tape®

OCCCLUDING and RETRACTING LOOP

DESCRIPTION:

Radiopaque silicone tubing, silicone adhesive, stainless steel needle and thermoplastic clamp.

INDICATIONS FOR USE:

Radiopaque silicone tubing with sealed ends to provide an air-cushioned surgical tool for manipulating anatomical organs and structures to either retract them or ligate vessels with minimum trauma.

WARNING:

Intended for single patient use only. DO NOT REUSE. After use, this product may be a potential biohazard. Handle and discard in accordance with acceptable medical practice and applicable local, state, and federal laws and regulations.

CAUTION:


Prior to use, examine the package carefully before opening to confirm its integrity and the expiration date has not passed. The device is supplied in a sterile package and is non-pyrogenic. Do Not Use if package is damaged, opened or the expiration date has passed.

Sterilized by Ethylene Oxide. **Do Not Resterilize.** RX only.

CONTRAINDICATIONS: None known.

INSTRUCTIONS FOR USE:

- A) Ensure distance 'D' is greater than length 'L'
- B) CAUTION: Do not use excessive force when pushing needle through locking mechanism.
- C) When using to occlude, slightly stretch tubing to tighten before sliding into slot of keyhole locking mechanism.

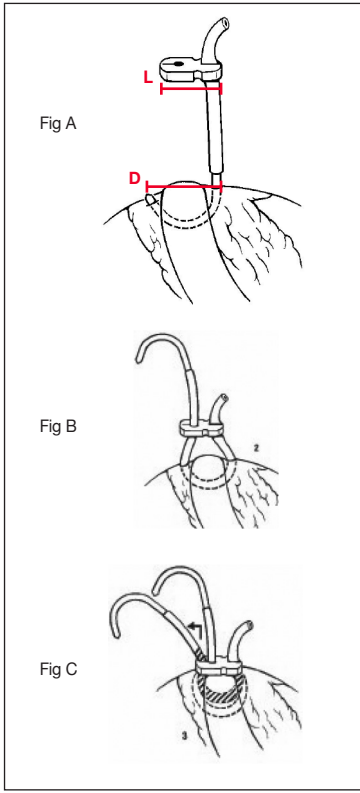REF	1541	1571
	1.2 mm (0.049 in.)	1.2 mm (0.049 in.)
	10cm (4 in.)	10cm (4 in.)
Needle Aiguille Ago Aguja Nadel	18Ga 	18Ga
Units Unités Unità Unidades Einheiten	 2 x 8 = 16	

For symbols glossary visit www.questmedical.com/resources.aspx

STERILE

EC REP

QUEST Medical, Inc.

One Allentown Parkway, Allen, Texas 75002 USA
+1.972.390.9800 | +1.800.627.0226
F +1.972.390.8465

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

© 2019 Quest Medical, Inc. All rights reserved.
Retract-O-Tape and SaddleLoop are registered trademarks
of Quest Medical, Inc.

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