Q2® INJECTION SITE

REF 3150 Approximate Priming Volume 0.09 mL

For symbols glossary visit www.questmedical.com/resources.aspx

QUEST Medical, Inc.

One Allentown Parkway | Allen, Texas 75002 USA
+1.972.390.9800 | +1.800.627.0226 | F +1.972.390.8465

©2019 Quest Medical, Inc. Q2 is a registered trademark of Quest Medical, Inc.

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Q2® INJECTION SITE Instructions For Use

INTENDED USE:

I.V. Administration set accessory that allows needleless luer connections to luer access devices.

CONTRAINDICATIONS:

None known.

WARNINGS:

- 1) Intended for Single Use only. DO NOT REUSE.

CAUTIONS:

- 1) Before using, examine the package carefully before opening to confirm its integrity and that the expiration date has not passed. The device is supplied in a sterile package and is non-pyrogenic. Do Not Use if package is damaged, opened or the expiration date has passed. Do not resterilize.
- 2) Do not use needles or blunt cannulae with the Q2® injection site.

PROCEDURE - USE ASEPTIC TECHNIQUE:

- 1) Swab septum with 70% alcohol antiseptic and allow to air dry.
- 2) Attach male luer connector at distal end of syringe or I.V. tubing into female luer of Q2 by pushing in and twisting clockwise until tight. Do not over tighten. For slip luers, push syringe into Q2 while twisting.

- 3) I.V. connection is ready for use. Deliver I.V. fluid or medication in usual manner.

DISCONNECTING I.V. LINES AND SYRINGES

- 4) Disconnect by twisting male luer connector at the distal end of I.V. tubing or syringe counter-clockwise until loose and apart. **DO NOT REMOVE Q2** from patient's catheter or tubing.
- 5) Flush Q2 per hospital protocol.
- 6) Q2 automatically closes and provides back check valve safety. Capping is not necessary.
- 7) To reconnect, repeat from step 1.
- 8) Replace per hospital protocol.

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