# 20 mm Q2® VIAL ADAPTER

**REF** 3160 Approximate Priming Volume 0.06 mL

For symbols glossary visit [www.questmedical.com/resources.aspx](http://www.questmedical.com/resources.aspx)

**QUEST Medical, Inc.**

One Allentown Parkway | Allen, Texas 75002 USA  
+1.972.390.9800 | +1.800.627.0226 | F +1.972.390.8465

©2019 Quest Medical, Inc. Q2 is a registered trademark of Quest Medical, Inc.

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# 20 mm Q2® VIAL ADAPTER Instructions For Use

### **INTENDED USE:**

Multiple needleless access to an injection medication vial for the purpose of facilitating the withdrawal or addition of solutions from or to the vial.

### **CONTRAINDICATIONS:**

None known.

### **WARNINGS:**

- 1) Intended for Single Use only. DO NOT REUSE.

### **CAUTIONS:**

- 1) Before using, examine the package carefully before opening to confirm its integrity and that the expiration date has not passed. The device is supplied in a sterile package and is non-pyrogenic. Do Not Use if package is damaged, opened or the expiration date has passed. Do not resterilize.
- 2) Do not use needles or blunt cannulae with the Q2® vial adapter.

### **PROCEDURE - USE ASEPTIC TECHNIQUE:**

- 1) Swab top of medication vial with 70% alcohol antiseptic. Allow to air dry, attach Q2 vial adapter to medication vial.
- 2) Swab septum with 70% alcohol antiseptic prior to accessing valve port.
- 3) Connect luer syringe to needle free valve port by pushing in and twisting clockwise until tight. Do not over tighten. For slip luers, push syringe into needle free port while twisting.
- 4) Withdraw or add desired fluid volume.

### **DISCONNECTING SYRINGES**

- 5) Disconnect by twisting luer lock syringe counter-clockwise until loose and apart. For slip luers, simply twist while pulling the syringe out of septum.
- 6) Q2 Vial Adapter automatically closes and provides back check valve safety. Capping is not necessary.
- 7) To reconnect syringe, repeat from step 2.
- 8) Discard vial adapter with empty medication vial.

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