

PRINTED MATERIAL SPECIFICATION

Spec. No. 902132 Rev. A

Issue Date OCT 30 1997 Supersedes 902132 Rev. NC Page 1 of 3

Product Description:

Unidirectional Purge Line, L-3

Code No. 4002104

A. Dimensions: 4 1/4" W x 5 1/2" H ± 1/8"

B. Description: Pouch Insert

1. Stock and color: White 20 lb. bond

2. Style: See following page(s)

3. Printing: Black

4. Manufacturer's Joint: None

5. Copy Requirements: See following page(s)

6. Other: Refer to Quest General Printed Material Spec. 900-01.

CONTROLLED COPY  
ONLY IF  
THIS STAMP IS

5

APPROVALS:

Kathy Schellenger 10-28-97  
Documentation Date

J. W. [Signature] 10/29/97  
Approval Date

Effective Date: OCT 30 1997



Front of Insert



**PURGE LINE**

**L-3**

---

CAT. NO. 4002104

1 UNIT

This line can be used to provide constant purge from pressurized chambers (e.g., Blood Filter, Bubble Trap, etc.). The one-way valve in the line is intended to prevent retrograde flow. The flow regulating valve can be used to adjust flow rates.

Contents of unopened, undamaged package are sterile/nonpyrogenic. Do not resterilize or reuse.

Store at dry controlled room temperature: 10°-25°C.

**SPECIFICATIONS:** FITTINGS: Male luer at each end. CHECK VALVE: One-way duckbill valve responds to pressure differentials. FLOW REGULATOR: Thumb controlled screw. LENGTH: 90cm approx.

---



---

**DIRECTIONS: USE ASEPTIC TECHNIQUE.**

1. Remove from the package.
2. Attach to apparatus as desired. Observe flow direction. TEST for proper function and flow.
3. Regulate purge flow rate as desired.
4. Periodically check for proper operation.
5. Discard after use.

PRINTED MATERIAL SPECIFICATION

Spec. No. 902132 Rev. A

Issue Date OCT 30 1997 Supersedes 902132 Rev. NC Page 3 of 3

Back of Insert

**CAUTION:** Federal (USA) law restricts this device to sale by or on the order of a physician.

---

**QUEST** Medical, Inc.

ALLEN, TEXAS 75002 USA / 800 627-0226

Made in USA

902132 Rev A  
Oct. 97