For prevention of retrograde flow when used with centrifugal pump. Contents of un-opened, undamaged package are sterile/non-pyrogenic. Do not resterilize or reuse.

DIRECTIONS: USE ASEPTIC TECHNIQUE.


1. Remove from the package and discard the end caps.
2. Connect the valve to the 3/8" arterial line proximal to the arterial line filter (anywhere between the venous reservoir outlet and the arterial filter) Post-operative recovery of arterial blood distal to the valve is facilitated by placing the valve within the filter bypass loop. The direction of flow is indicated on the valve by arrows.

WARNING: The valve must be oriented properly with the direction of blood flow. Valve malfunction and possible damage to blood components will result if the valve is positioned incorrectly.

3. The valve must be primed in the forward flow direction.
4. When priming, hold the valve with the outlet end pointing up to facilitate the removal of air.
5. The valve and filter must be drained in the forward flow direction.

CAUTION: 1) This device is not intended for use at pressures above 30 psi nor for periods of use exceeding 6 hours. 2) If RetroGuard valve is placed proximal to arterial filter purge line, retrograde flow through the purge line may result.

FOLLOW INSTRUCTIONS CAREFULLY: The surgical team is solely responsible for determining the suitability of the valve relative to the CPB circuit, equipment, and surgical technique employed.

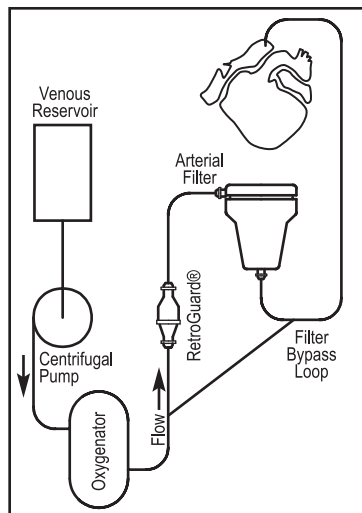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©2019 Quest Medical, Inc. RetroGuard is a registered trademarks of Quest Medical, Inc.