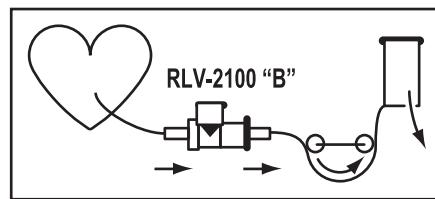
**RLV-2100 "B"****REF 4103102****1 UNIT****SUCTION CONTROL VALVE,
NON-ADJUSTABLE (For L. V. Venting)****INSTRUCTIONS:**

- 1) Peel pouch open using aseptic technique. Discard protective nipples.
- 2) Insert valve into 1/4 -inch I.D. vent line (either near the perfusionist's roller pump or on the sterile field) at least 60 cm from the heart and preferably level with it, within view and reach of the surgical team.
- 3) Arrows on valve indicate flow direction.
Cardioplegia line should be clamped off when vacuum line is activated. Vacuum line should be closed off when cardioplegia line is activated. Not clamping the cardioplegia line when activating vacuum line may cause positive pressure build up resulting in positive pressure relief to the atmosphere through the relief band.

WARNING: Air and blood venting out of the side of the valve indicate a problem which must be rectified immediately.

Keep at dry/cool temperature.

For symbols glossary visit www.questmedical.com/resources.aspx

STERILE EO **Rx ONLY****STERILE:** If package is unopened or undamaged.**STERILE :** Tant que la pochette est fermée/hon endommagée**ESTÉRIL:** cuando el paquete no ha sido abierto y no está dañado.**STERIL:** Før posen åbnes/ikke er beskadiget.**STERIEL:** Als het pakket ongeopend/ onbeschadigd is.**STERIL:** bei ungeöffneter/unbeschädigter Packung.**STERILE:** purché la confezione non sia stata aperta o danneggiata.

U.S. Patent #4,758,224

**QUEST** Medical, Inc.

One Allentown Parkway | Allen, Texas 75002 USA
+1.972.390.9800 | +1.800.627.0226 | F +1.972.390.8465

©2020 Quest Medical, Inc. All rights reserved.

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