RLV-T

REF

4103105

1 UNIT

SUCTION CONTROL VALVE  
(For L. V. Venting)

INSTRUCTIONS:

- 1) Peel pouch open using aseptic technique. Discard protective nipples.
- 2) Insert valve into 1/4 -inch I.D. vent line (either near the perfusionist's roller pump or on the sterile field) at least 60 cm from the heart and preferably level with it, within view and reach of the surgical team. Arrows on valve indicate flow direction.
- 3) Maximum flow may be achieved by clamping the side tube closed. This disables the negative pressure relief feature of the device.
- 4) Cardioplegia line should be clamped off when vacuum line is activated. Vacuum line should be closed off when cardioplegia line is activated. Not clamping the cardioplegia line when activating vacuum line may cause positive pressure build up resulting in positive pressure relief to the atmosphere through the relief band.

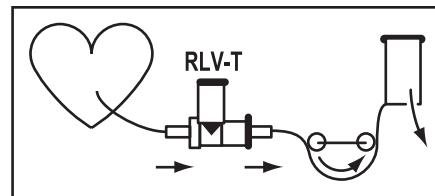
**WARNING:** Air and blood venting out of the side of the valve indicate a problem which must be rectified immediately.

For symbols glossary visit [www.questmedical.com/resources.aspx](http://www.questmedical.com/resources.aspx)

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QUEST Medical, Inc.

One Allentown Parkway | Allen, Texas 75002 USA  
+1.972.390.9800 | +1.800.627.0226 | F +1.972.390.8465

©2019 Quest Medical, Inc. All rights reserved.

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