

CAT. NO. 5001007

DESCRIPTION

Isolation transformer module bracket and hardware.

INDICATION FOR USE

The MPS® isolation transformer module bracket is indicated to be used to affix the Quest MPS isolation transformer module to the heart lung machine or MPS stand alone cart masthead.

CONTRAINDICATION

This product is designed and warranted for use with the Quest MPS myocardial protection system only. Use with other systems is contraindicated. This product is not designed, sold, or intended for use except as indicated.

WARNINGS AND CAUTIONS


Read instructions carefully. The attending clinician is solely responsible for the setup and use of this device.

For use with Quest's MPS system only. All claims void if used separately.

Do not over-tighten rigid connections.

INSTRUCTIONS FOR USE

1. Remove the MPS ITM bracket from its packaging.
2. Clamp the bracket firmly to the masthead on the heart lung machine or MPS stand alone cart so that the isolation transformer module will be in close proximity to the MPS console.
3. Place MPS isolation transformer module onto shelf of the bracket and secure underneath with the 4 screws and washers provided.
4. Refer to the MPS console Operations Manual for further instructions.

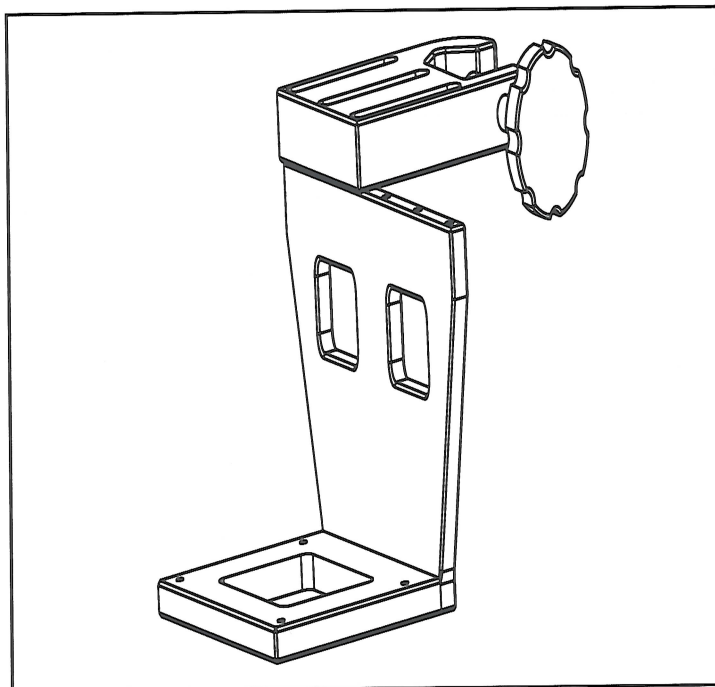

*The MPS® and MPS® Sterile Disposables are covered
under one or more of the following U.S. patents:*

5,385,540
5,573,502
5,588,816

5,638,737
5,645,531
Patents Pending