

Label Spec No. 902574Revision: B

Product Description:
MPS® Console Adapter (S-7000)

Product Code: 5001009

- A. Dimensions: 8 ½" W x 11" H
- B. Description: 5001009, Carton Insert

Stock and color: White 20Lb. Bond

- 1. Style: See following page
- 2. Printing: Black Ink
- 3. Manufacture's Joint: None
- 4. Copy Requirements: See following pages
- 5. Quest Inspection Procedure: 900-01

Any additional comments: None

Label Spec No. 902574 revB

REVISION HISTORY

REV	DESCRIPTION OF CHANGE	BY	DATE	ECO
A	New Document	Unk	6/24/98	Unk



MPS® CONSOLE ADAPTER (S-7000)

CAT. NO. 5001009

DESCRIPTION

MPS® Console Adapter (S-7000).

INDICATION FOR USE

The MPS console adapter is indicated to be used to affix the Quest MPS console to a heart lung machine console.

CONTRAINDICATION

This product is designed and warranted for use with the Quest MPS myocardial protection system only. Use with other systems is contraindicated. This product is not designed, sold, or intended for use except as indicated.


WARNINGS AND CAUTIONS

Read instructions carefully. The attending clinician is solely responsible for the setup and use of this device.

For use with Quest's MPS system only. All claims void if used separately.

INSTRUCTIONS FOR USE

1. Remove MPS console adapter from its packaging.
2. Position console adapter in vacated roller pump slot. Ensure end blocks are facing upward and slotted opening in base plate is positioned over pin extending from 7000 console. When console adapter is installed in any interior position on heart-lung-machine (HLM), adapter is to be oriented so that label is at rear of console. If console

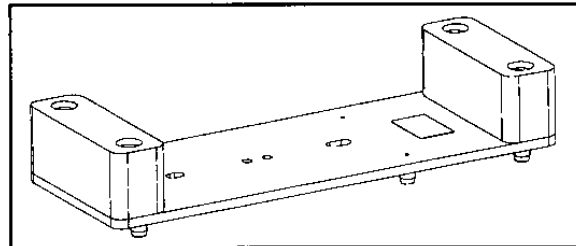


is to be used in either end position on HLM, label should be at front of console.


3. Secure console adapter with existing screw knob on 7000 console.
4. Place MPS console on adapter plate by inserting MPS console's feet in four shallow recesses on top surface of end blocks.
5. Remove 7000 adapter gaskets from packaging.

NOTE: When MPS console is to be used in leftmost position on HLM only right adapter gasket is to be installed. When used in rightmost position on HLM only left adapter gasket is to be installed. When MPS console is installed in any interior position on HLM, both adapter gaskets are to be installed.

6. To install left adapter gasket, insert gasket in the gap between MPS and HLM and position on left side of MPS console as shown in appropriate illustration. Lower edge of gasket should rest on the surface of HLM and angled surface of gasket should line up with matching angle on console side panel. Press gasket firmly into position.
7. If MPS console is to be used in either end position on HLM, gasket should be positioned as shown in other drawing so that back edge of gasket extends ¼" past back edge of side panel.
8. Install right adapter gasket, if required, in same manner as above.
9. Refer to MPS console Operations Manual for further instructions.

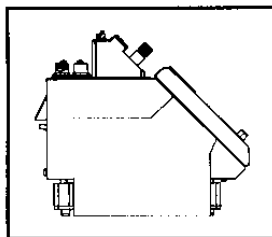

INTERIOR POSITIONS

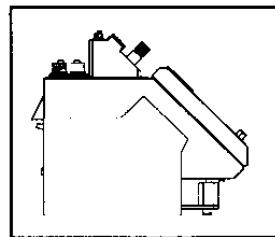

RIGHT END POSITIONS

The MPS® and MPS® Sterile Disposables are covered under one or more of the following U.S. patents: 5,385,540 • 5,638,737 • 5,573,502 • 5,645,531 • 5,588,816 • Patents Pending

QUEST Medical, Inc.
An **Atrion** company

ONE ALLENTOWN PARKWAY / ALLEN, TEXAS 75002 USA
800 627-0226 / 972 390-9800 / FAX: 972 390-8465

902574 REV. A
APR. 98