

### Intended Use:

The MPS<sup>®</sup>2 Locking Block is intended to facilitate the installation and use of a 10 Convolution Heat Exchanger or a 16 Convolution Heat Exchanger.

### Contraindications:

This device is designed for use with the Quest MPS 2 console and Disposables. Use with other systems is contraindicated. This device is not designed, sold, or intended for use except as indicated.

### Warnings and Cautions:

1. Read instructions carefully. The attending clinician is solely responsible for the setup and device use. See MPS System Operations Manual for complete instructions for use.
2. For use ONLY with the MPS console. DO NOT USE for any other purpose. All claims void if used separately.
3. Unpack the three packages and inspect for any shipping damage. DO NOT USE damaged components.
4. The hospital-grade electrical outlet or appropriate hospital outlet should have proper grounding.
5. Federal (USA) law restricts this device to sale by or on the order of a physician.
6. Observe all additional warnings and cautions contained in the following Instructions For Use.

### To Actuate to the 16 Convolution Heat Exchanger Position



Start from the 10 C unlocked position.



1. Push down on the knob and rotate it counter-clockwise 360 degrees until it stops rotating. (Locking knob will extend outward)



2. The knob is now in the 16 Convolution Unlocked Position.

(over)

For symbols glossary visit [www.questmedical.com/resources.aspx](http://www.questmedical.com/resources.aspx)



One Allentown Parkway, Allen, Texas 75002 USA  
+1.972.390.9800 | +1.800.627.0226 | F +1.972.390.8465

© 2019 Quest Medical, Inc. MPS is a registered trademark of Quest Medical, Inc. The MPS Console and MPS sterile disposables are covered under one or more of the following U.S. patents: 5,385,540 . 5,573,502 . 5,588,816 . 5,638,737 . 5,645,531 . 5,899,873. Also covered by pending U.S. and International Patents and patent applications.



EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

**Made in USA**  
904506 REV B  
MAR 19

## To Actuate to the 10 Convolution Heat Exchanger Position



Start from the 10 C unlocked position.



1. Pushing down on the knob, rotate the knob 360 degrees clockwise until it clicks into place.



2. The rounded indentation on the two pieces of the knob should be lined up now.



3. The knob is now in the 10 Convolution Unlocked Position.

- If you are having issues locking down a heat exchanger, ensure the locking block is in the correct configuration for the disposable set you wish to install.
- Please refer to the *Instructions for Use* supplied with the disposable set for installation of the disposable set.
- When finished using this IFU as a reference sheet, please attach in the back of your MPS Operations Manual in the updates section.
- Product Code: 5161102 16 Convolution Delivery Set

For symbols glossary visit [www.questmedical.com/resources.aspx](http://www.questmedical.com/resources.aspx)



One Allentown Parkway, Allen, Texas 75002 USA  
+1.972.390.9800 | +1.800.627.0226 | F +1.972.390.8465

© 2019 Quest Medical, Inc. MPS is a registered trademark of Quest Medical, Inc. The MPS Console and MPS sterile disposables are covered under one or more of the following U.S. patents: 5,385,540 . 5,573,502 . 5,588,816 . 5,638,737 . 5,645,531 . 5,899,873. Also covered by pending U.S. and International Patents and patent applications.



EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

**Made in USA**  
904506 REV B  
MAR 19