REF

511800A MPS[®] Aortic Transducer Interface Cable

511800R MPS Retrograde Transducer Interface Cable

5140004 MPS Pole Mount Transducer Holder

DESCRIPTION

Aortic transducer interface cable with red collared male connector.
Cable connects a disposable pressure transducer to the MPS console.

Retrograde transducer interface cable with blue collared male connector.
Cable connects a disposable pressure transducer to the MPS console.

MPS Pole mount holder allows the transducer to attach to the mast of the MPS console Heart lung machine (HLM).

INDICATION FOR USE

The transducer interface cable is indicated to be used as the electronic interface between the MPS console and a disposable pressure transducer for purpose of monitoring delivery pressure.

CONTRAINDICATION

This device is designed for use with only the Quest MPS and MPS 2 console. Use with other products is contraindicated. This device is not designed, sold, or intended for use except as indicated.

WARNINGS and CAUTIONS

1. Read instructions carefully. The attending clinician is solely responsible for the setup and use of this device.
2. Use cable to interface disposable pressure transducer with the MPS console.
3. The transducer will not accurately measure pressures above 300 mmHg.
4. Do not immerse electrical connectors. Do not autoclave, or kink cables.

INSTRUCTIONS FOR USE

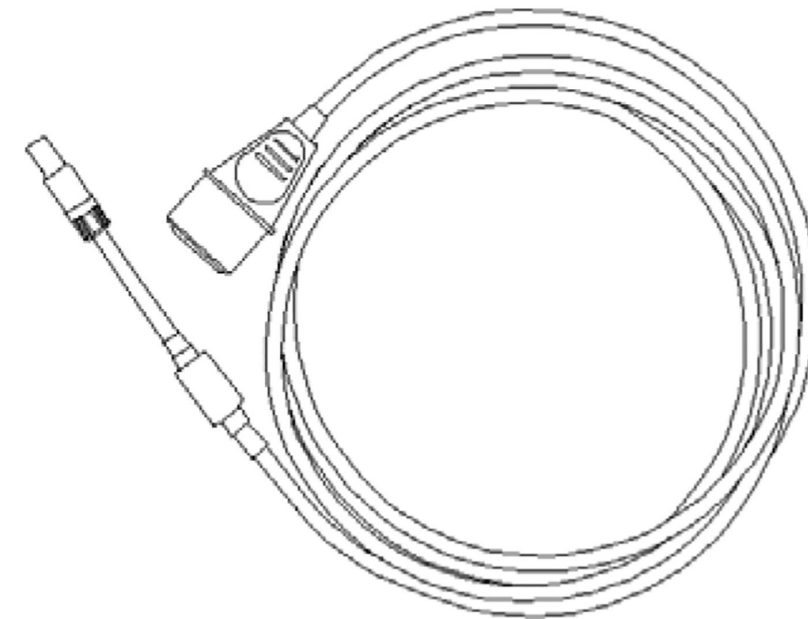
1. Connect transducer end of MPS interface cable to the disposable transducer cable connector.
2. Plug MPS interface cable connector into the matching letter A or R port on back of the MPS console. Antegrade/Aortic (A) is color codes Red. Retrograde is color coded Blue.
3. Follow disposable pressuring transducer setup procedure to ensure proper priming and zeroing.

REF

511800A MPS[®] Aortic Transducer Interface Cable

511800R MPS Retrograde Transducer Interface Cable

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