

REF 513UEXT MPS[®] 3 ECG Extension Cable

1 UNIT



DESCRIPTION:

Cable with one 1/4 in. phone plug and one 1/4 in. phone jack. Connects MPS[®] 3 Controller to ECG monitor, as an extension of the MPS 3 ECG interface cable.

INDICATION FOR USE

The ECG interface cable is indicated to be used as the electronic interface between the MPS 3 system and an ECG monitor for the purpose of replicating the ECG trace on the MPS 3 Controller.

CONTRAINDICATION

This product is designed and warranted for use with the Quest MPS 3 cardioplegia delivery system only. Use with other systems is contraindicated. This product is not designed, sold, or intended for use except as indicated.

WARNINGS and CAUTIONS

1. Read instructions carefully. The attending clinician is solely responsible for the setup and use of this device.
2. Use cable to connect MPS 3 Controller to listed ECG monitor so that the ECG trace is replicated (displayed) on the Controller screen.
3. Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.
4. Do not immerse electrical connectors.
Do not autoclave or kink cables.

INSTRUCTIONS FOR USE

1. Connect the 1/4 in. phone plug to the ECG monitor.
The orientation does not matter.
2. Connect one 1/4 in. phone plug from the MPS 3 ECG cable into the 1/4 in. phone jack and the other into the MPS 3 Controller.
3. Power on the ECG and MPS 3 systems.
4. Follow instructions in the MPS 3 user's manual for displaying the ECG trace.