



REF 513USCC MPS® 3 RS232 Cable

1 UNIT



DESCRIPTION:

MPS®3 RS232 Cable. Use this cable to connect the MPS 3 Controller to an electronic medical data/records system.

INDICATION FOR USE

The RS232 cable is indicated to be used as the electronic interface between the MPS 3 console and an electronic medical records system for the purpose of data transfer.

CONTRAINDICATION

This product is designed and warranted for use with the Quest MPS 3 cardioplegia delivery system only. Use with other systems is contraindicated. This product is not designed, sold, or intended for use except as indicated.

WARNINGS and CAUTIONS

1. Read instructions carefully. The attending clinician is solely responsible for the setup and use of this device.
2. Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.
3. Do not immerse electrical connectors. Do not autoclave or kink cables.

INSTRUCTIONS FOR USE

1. Plug the male end of the cable into the MPS 3 Controller.
2. Plug the female end of the cable into an electronic medical records system.
3. Follow the instructions in the MPS 3 user's manual to initiate data transfer.



QUEST Medical, Inc.

ONE ALLENTOWN PARKWAY | ALLEN, TEXAS 75002 USA
+1.972 390-9800 | +1.800 627-0226



Made in USA
904553 REV A
NOV 2018