



16 in. (38 cm) Q2® CheckMate® Extension Set

With 3 pressure activated checkvalves and inline checkvalve
This device is not made with the plasticizer Diethylhexyl Phthalate (DEHP)

REF

9548 Approximate priming volume 3.5 mL

1 UNIT

Intended Use:


For administration of intravenous fluids to a patient's vascular system utilizing needle-free components and IV manifold for multiple simultaneous intravenous therapy via gravity, syringe, or infusion pump. Use of needleless system may aid in the prevention of needle stick injuries.

Procedure — Use Aseptic Technique:

1. Place pouch on surface and remove set.
2. Ensure all connections and caps are tightened as necessary.
3. Drip chamber sets: Close clamp, if applicable, on set. Prepare solution container. Insert spike and suspend container on IV pole. Squeeze and release drip chamber until it is approximately half full. Open clamp and allow fluid to fill tubing.
4. Non-drip chamber sets: Prime set by attaching main IV set to top most port on IV set and allow fluid to fill entire set.
5. NOTE: Entrained air may be removed from manifold, Y-sites, and inline checkvalve by inverting and tapping. Q2 injection sites may be primed using a flush syringe.
6. Close clamp.
7. Connect set to patient's vascular access device and establish flow.

(OVER)



 **For Warnings & Cautions**
please consult:
www.questmedical.com

Rx ONLY



QUEST Medical, Inc.

Made in U.S.A.
904499 REV C
MAY 2018

One Allentown Parkway • Allen, TX 75002 • +1 972 390-9800 • +1 800 627-0226 • www.questmedical.com
©2018 Quest Medical, Inc. CheckMate and MultiPort are registered trademarks of Quest Medical, Inc.

Secondary Infusions and Preparation of Q2® Injection Sites:

Manifolds have inline gravity checkvalves on top most port and one-way injection ports in the following configurations:

- Q2 CheckMate® side ports: Pressure activated checkvalves (infusion via syringe or pump)
- Q2 MultiPort® side ports: Gravity checkvalves (infusion via gravity, syringe or pump)

1. Prime a secondary set or syringe in the usual manner.
2. If necessary, swab septum with preferred antiseptic and allow to dry, or remove injection site cap.
3. Expel air as needed.
4. Attach secondary set and infuse desired amount of fluid.
5. Replace cap if necessary.

IV Push:

1. Prepare syringe.
2. Close clamp between injection site and vascular access device. Attach syringe. DO NOT allow air to enter line.
3. Open clamp downstream of injection site.
4. Push medication as prescribed and flush tubing.