



QLINCH® IV Securement Band

REF

QIVB0060

QLINCH® IV Securement Band

60 UNITS

Product Description:

The QLINCH® IV Securement Band is a disposable, flexible, non-adhesive device that provides stable, secondary securement of IV tubing and minimizes the need for taping tubing to the skin. The unique, self-adjusting design provides the ideal forces/tension to safely hold IV tubing to patient's extremities and is designed to help reduce catheter movement and displacement.

Intended Use/Indication For Use:

The QLINCH® IV Securement Band is intended to help secure IV tubing to a patient's extremities.

Contraindications:

The device is not indicated for pediatrics.

Not for use on the hands or feet.

Not for use on non-intact skin.

WARNINGS:

This device is intended for single patient use only.

Removed Qlinch sections present risk of choking/suffocation when ingested. Discard removed sections immediately.

DO NOT place the QLINCH® Band directly over the IV hub, connectors, or inserted cannula.

DO NOT USE AS A TOURNIQUET. NEVER attach a QLINCH® Band by tying, taping, clamping, or by any means other than the engineered, self-adjusting segments.

If swelling of the extremity occurs, the QLINCH® Band will automatically readjust to the correct tension to prevent constriction. However, the device will not automatically retighten. If a reduction in limb swelling occurs, ensure that the band remains secure.



While the QLINCH® Band is designed to be self-adjusting to eliminate any risk of constriction of blood flow, patient's limbs should be monitored for signs of tourniqueting.

Instructions for use:

1. After the vascular access device and IV administration set have been secured at the insertion site (primary IV securement), apply a QLINCH® Band around the arm proximal to the insertion site to help secure the IV tubing.

- DO NOT apply distal to the wrist.
- DO NOT place the QLINCH® Band directly over the IV hub, connectors, or inserted cannula.

2. To apply, grasp the QLINCH® Band by the end tab and thread the tab through the slot on the opposite end of the band.

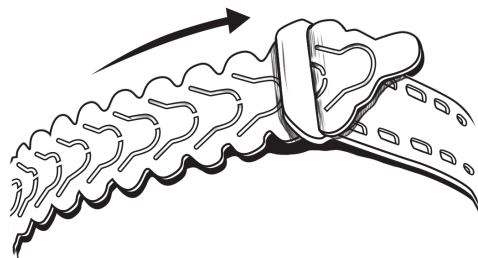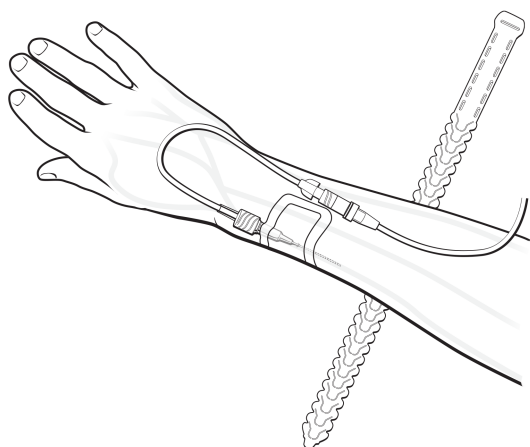

For symbols glossary visit www.questmedical.com/resources.aspx

(OVER)



QUEST Medical, Inc.


One Allentown Parkway, Allen, TX 75002 U.S.A. • +1 972 390-9800 • +1 800 627-0226 • www.questmedical.com
©2022 Quest Medical, Inc. QLINCH is a registered trademark of Quest Medical, Inc.




www.questmedical.com/elFU

Made in U.S.A.
904659 REV A
AUG 22


3. Pull the tab through the slot, holding the slot end in place as needed.

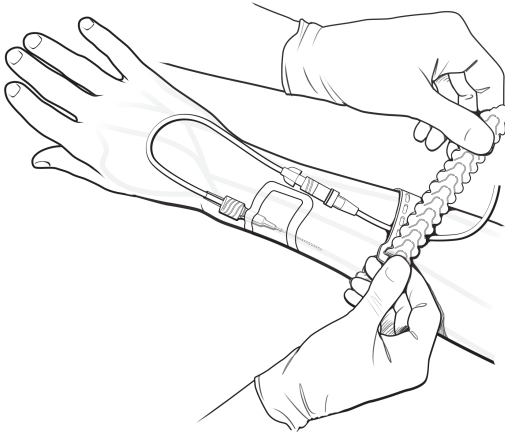


4. Continue to pull on the band while holding the slot end until the band is snug. The QLINCH® Band should self-adjust, loosening to the appropriate tension.

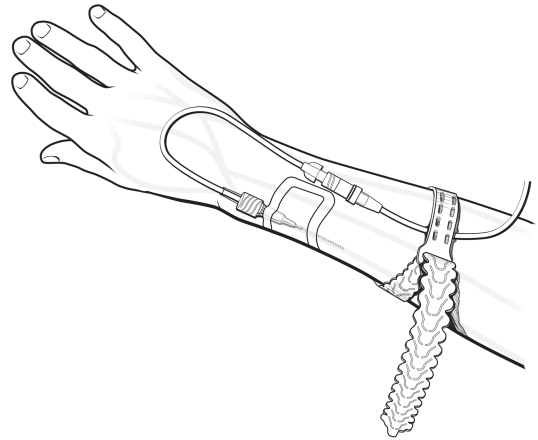


5. To shorten the length of the unused portion of the band, select a sufficiently long location along the band (4 or 5 segments) and hold the band securely at that location.

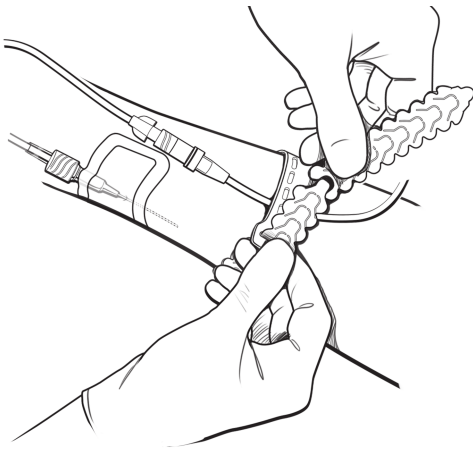

6. Pull off the unneeded segments. Note that removal of segments creates a new end tab, allowing for repositioning if needed.

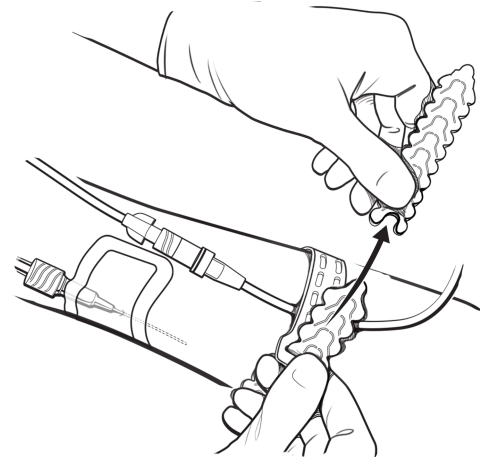

7. To remove, pull the band back through the slot.

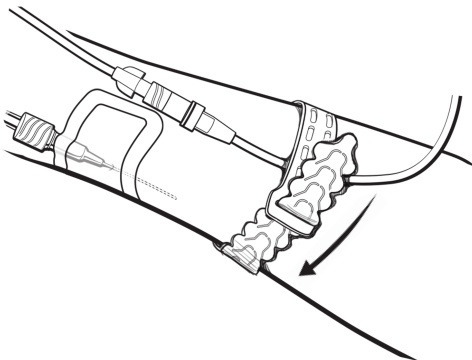

For symbols glossary visit www.questmedical.com/resources.aspx



Q QUEST Medical, Inc.

One Allentown Parkway, Allen, TX 75002 U.S.A. • +1 972 390-9800 • +1 800 627-0226 • www.questmedical.com
©2022 Quest Medical, Inc. QLINCH is a registered trademark of Quest Medical, Inc.



www.questmedical.com/eIFU

Made in U.S.A.
904659 REV A
AUG 22